



reddot winner 2023



CARE PUMP


EXPERT8
Advanced line

Number of chambers	8
Max. pressure [mmHg]	220
Number of proprietary therapeutic pre-programs of the medical organization	13
Features for visually impaired / blind people	voice assistant + special keyboard
Smart Gradient' function	mmHg, %, final pressure
Hold function [s]	1-50
Interval function [s]	1-50
Precise regulation of pressure and time	in steps of 1, 5 or 10
Setting of individual pressure in each chamber	included
Number of independent channels	1
Adjustment of the speed of the treatment	1-5
Cuffs	overlapping system
User's own programs	included
List of "Favourites" programs	included
Display	7" TN lcd color touch screen
Weight [kg]	3,6
Dimensions [mm]	300x260x160
Power supply	100÷240 V, 50÷60 Hz
Power consumption	40-70 W
Warranty	1 year

Accessories




Included:

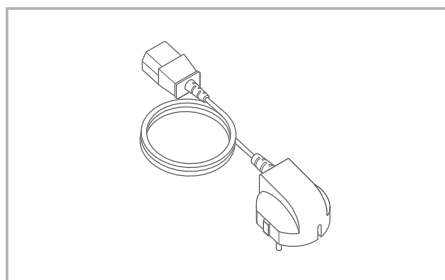


power supply

Optional:

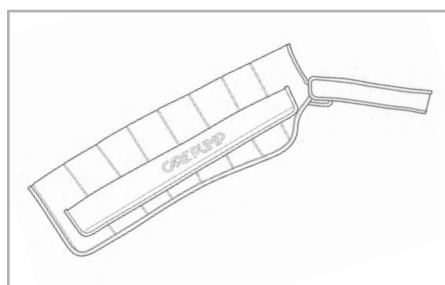


8-chamber arm cuff

CPARM8T

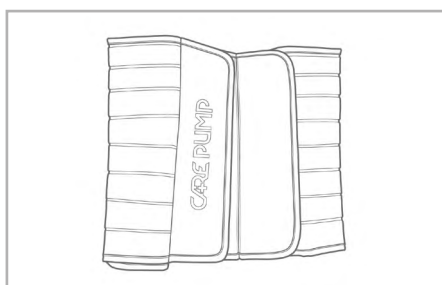


8-chamber waist cuff

CPWAIST8T

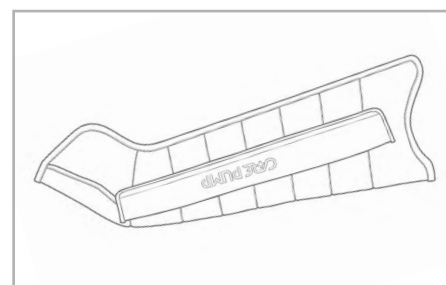


8-chamber left leg cuff

CPLEG8TL

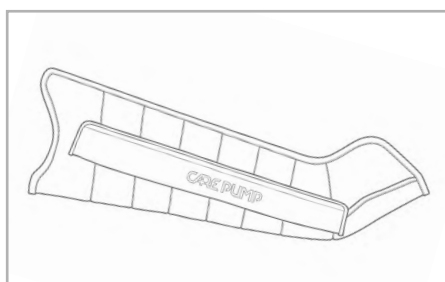


8-chamber right leg cuff

CPLEG8TR

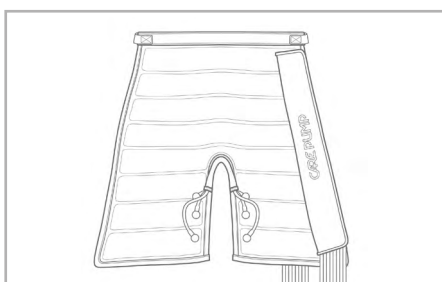


8-chamber short pants

CPSHORTPANTS8T

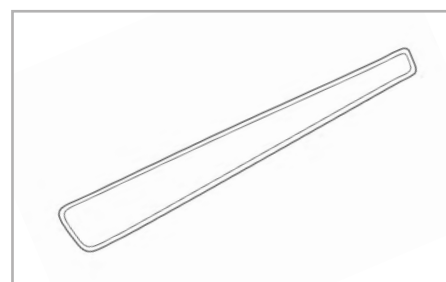


Extension zipper

CPLEGEXT8

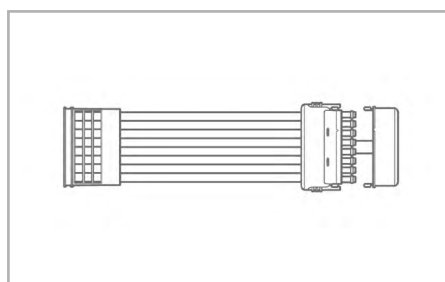


Double splitter

CPDOUBLECONNECTOR8

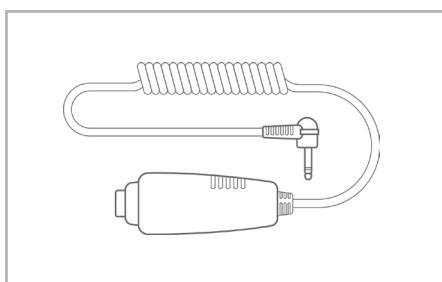


Patient's safety button

SAFETY BUTTON

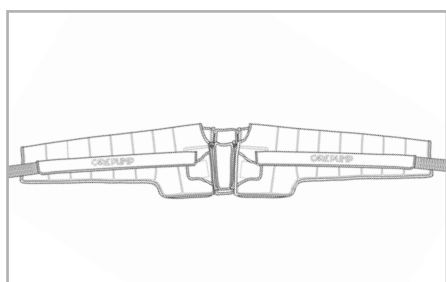


8-chamber double cuff for arms, shoulders and chest

CPDOUBLEARM8T

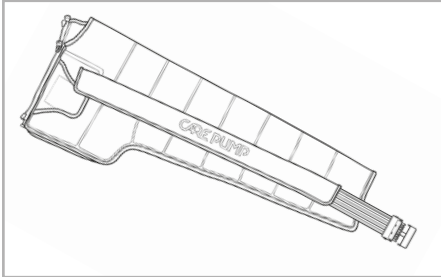


8-chamber left arm, shoulder and chest cuff

CPARMSB8TL

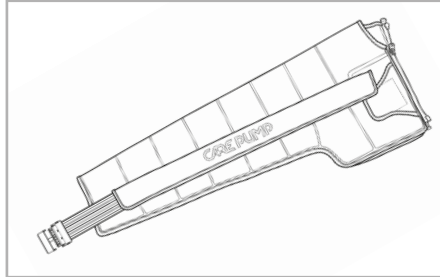


8-chamber right arm, shoulder and chest cuff

CPARMSB8TR

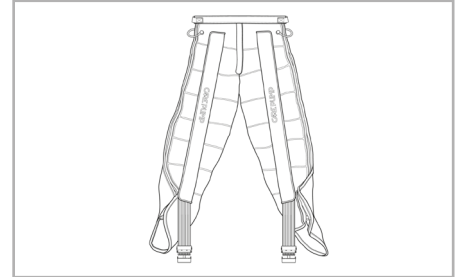


8-chamber full pants

CPLONGPANTS8T

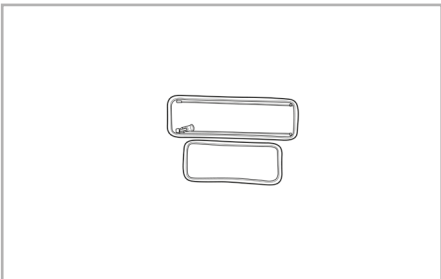


2 x Extension zippers for 8-chamber double cuff for arms, shoulders and chest

CPDOUBLEARMEXT8

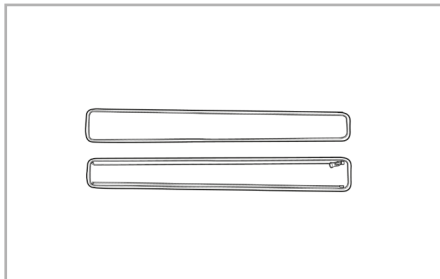


2 x Extension zippers for 8-chamber short pants

CPSHORTPANTSEXT8

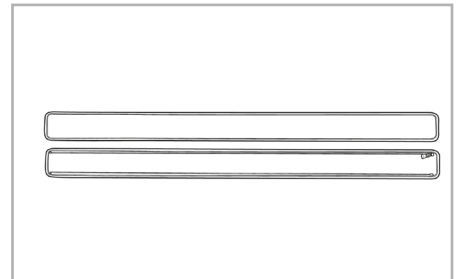

2 x Extension zippers for 8-chamber full pants

CPLONGPANTSEXT8

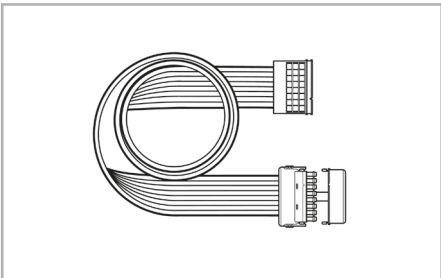

Double splitter (long - 150 cm) for 8-chamber devices

CPDOUBLECONNECTOR8L

Therapeutic pre-programs

of the medical organization



lymphoedema I

Gradient	1 mmHg
Time	50 min.
Hold	3 s
Interval	3 s

lymphoedema II

Gradient	1 mmHg
Time	50 min.
Hold	3 s
Interval	3 s

lymphoedema III

Gradient	1 mmHg
Time	50 min.
Hold	3 s
Interval	0 s

lymphoedema IV

Gradient	1 mmHg
Time	50 min.
Hold	3 s
Interval	0 s

venous oedema I

Gradient	3 mmHg
Time	45 min.
Hold	2 s
Interval	3 s

venous oedema II

Gradient	3 mmHg
Time	45 min.
Hold	2 s
Interval	2 s


lipodema I

Gradient	2 mmHg
Time	60 min.
Hold	3 s
Interval	3 s

lipodema II


Gradient	2 mmHg
Time	60 min.
Hold	3 s
Interval	3 s

post-injury therapy

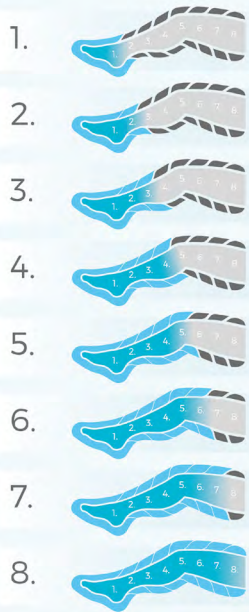


Gradient	4 mmHg
Time	30 min.
Hold	2 s
Interval	2 s

prevention, relaxation

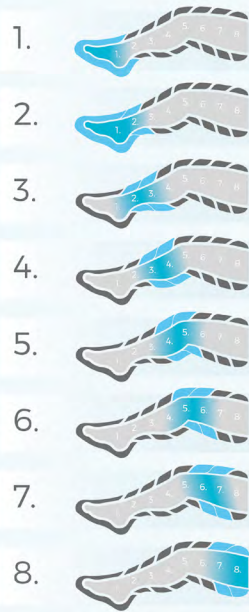


Gradient	0 mmHg
Time	60 min.
Hold	2 s
Interval	2 s

cellulite **advanced**

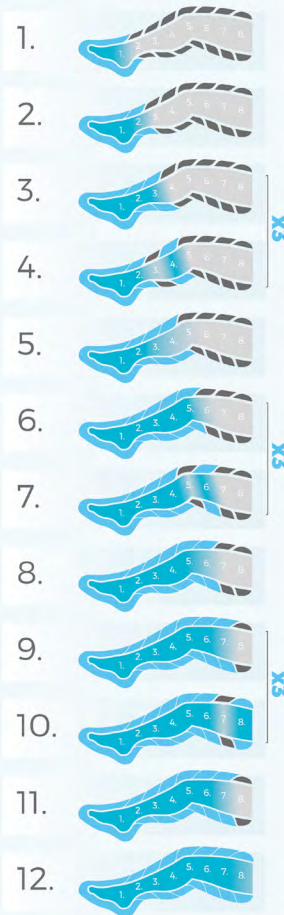


Gradient	1 mmHg
Time	60 min.
Hold	1 s
Interval	2 s

regeneration

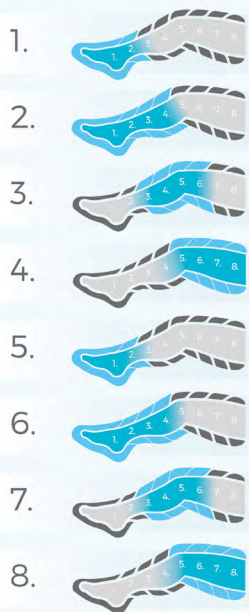

Gradient	4 mmHg
Time	45 min.
Hold	2 s
Interval	2 s

all-chamber massage

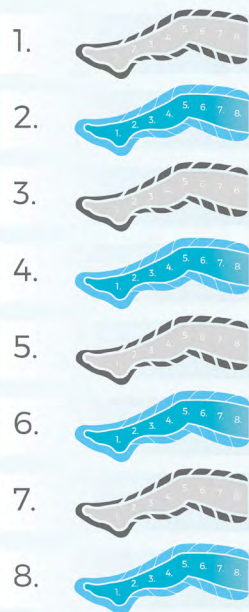

Gradient	0 mmHg
Time	30 min.
Hold	2 s
Interval	3 s