




## EXPERT 8<sup>DUO</sup> Advanced line

|                                                                            |                                    |
|----------------------------------------------------------------------------|------------------------------------|
| Number of chambers                                                         | 8                                  |
| Max. pressure [mmHg]                                                       | 220                                |
| Number of proprietary therapeutic pre-programs of the medical organization | 13                                 |
| Features for visually impaired / blind people                              | voice assistant + special keyboard |
| Smart Gradient' function                                                   | mmHg, %, final pressure            |
| Hold function [s]                                                          | 1-50                               |
| Interval function [s]                                                      | 1-50                               |
| Precise regulation of pressure and time                                    | in steps of 1, 5 or 10             |
| Setting of individual pressure in each chamber                             | included                           |
| Number of independent channels                                             | 2                                  |
| Adjustment of the speed of the treatment                                   | 1-5                                |
| Cuffs                                                                      | overlapping system                 |
| User's own programs                                                        | included                           |
| List of "Favourites" programs                                              | included                           |
| Display                                                                    | 9" TN lcd color touch screen       |
| Weight [kg]                                                                | 5,5                                |
| Dimensions [mm]                                                            | 385x310x180                        |
| Power supply                                                               | 100÷240 V, 50÷60 Hz                |
| Power consumption                                                          | 60-120 W                           |
| Warranty                                                                   | 1 year                             |

# Accessories




## Included:

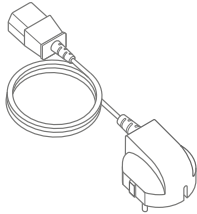


power supply

## Optional:

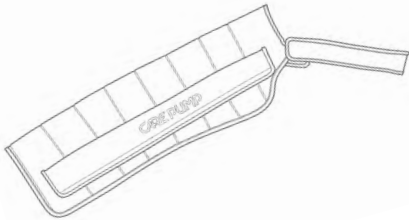


8-chamber arm cuff

CPARM8T

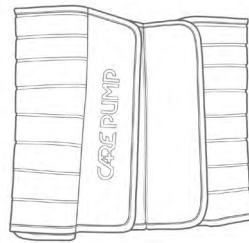


8-chamber waist cuff

CPWAIST8T

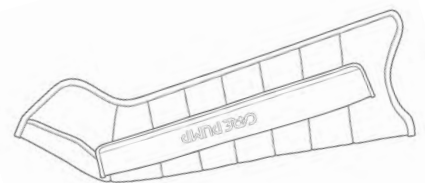

8-chamber left leg cuff

CPLEG8TL

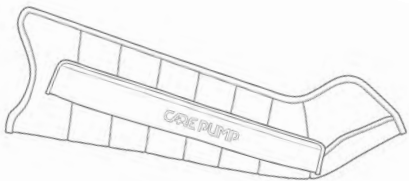

8-chamber right leg cuff

CPLEG8TR



8-chamber short pants

CPSHORTPANTS8T




Extension zipper

CPLEGEXT8




Double splitter

CPDOUBLECONNECTOR8

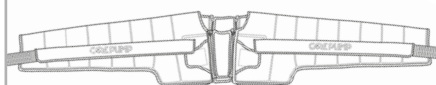


8-chamber double cuff for arms, shoulders and chest

CPDOUBLEARM8T

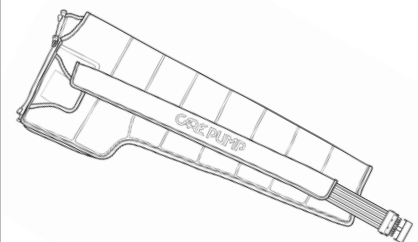


8-chamber left arm, shoulder and chest cuff

CPARMSB8TL

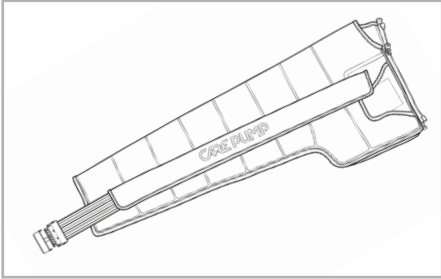


8-chamber right arm, shoulder  
and chest cuff

CPARMSB8TR

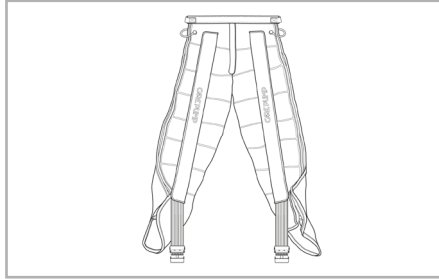


8-chamber full pants

CPLONGPANTS8T

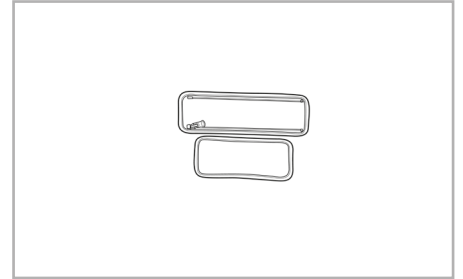


2 x Extension zippers  
for 8-chamber double cuff for  
arms, shoulders and chest

CPDOUBLEARMEXT8

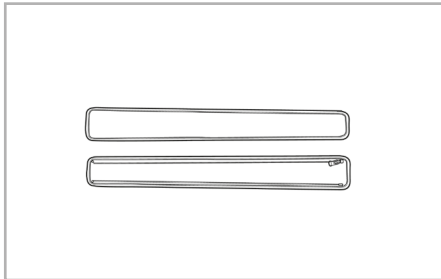


2 x Extension zippers  
for 8-chamber short pants

CPSHORTPANTSEXT8

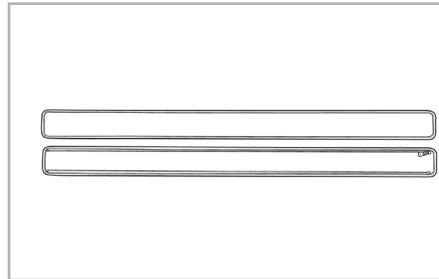

2 x Extension zippers  
for 8-chamber full pants

CPLONGPANTSEXT8

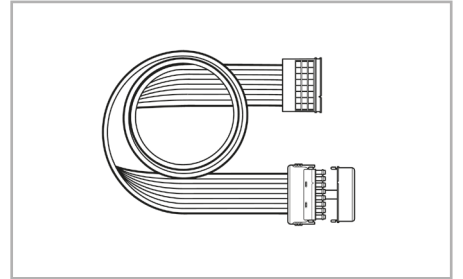


Double splitter (long - 150 cm)  
for 8-chamber devices


CPDOUBLECONNECTOR8L

# Therapeutic pre-programs

of the medical organization

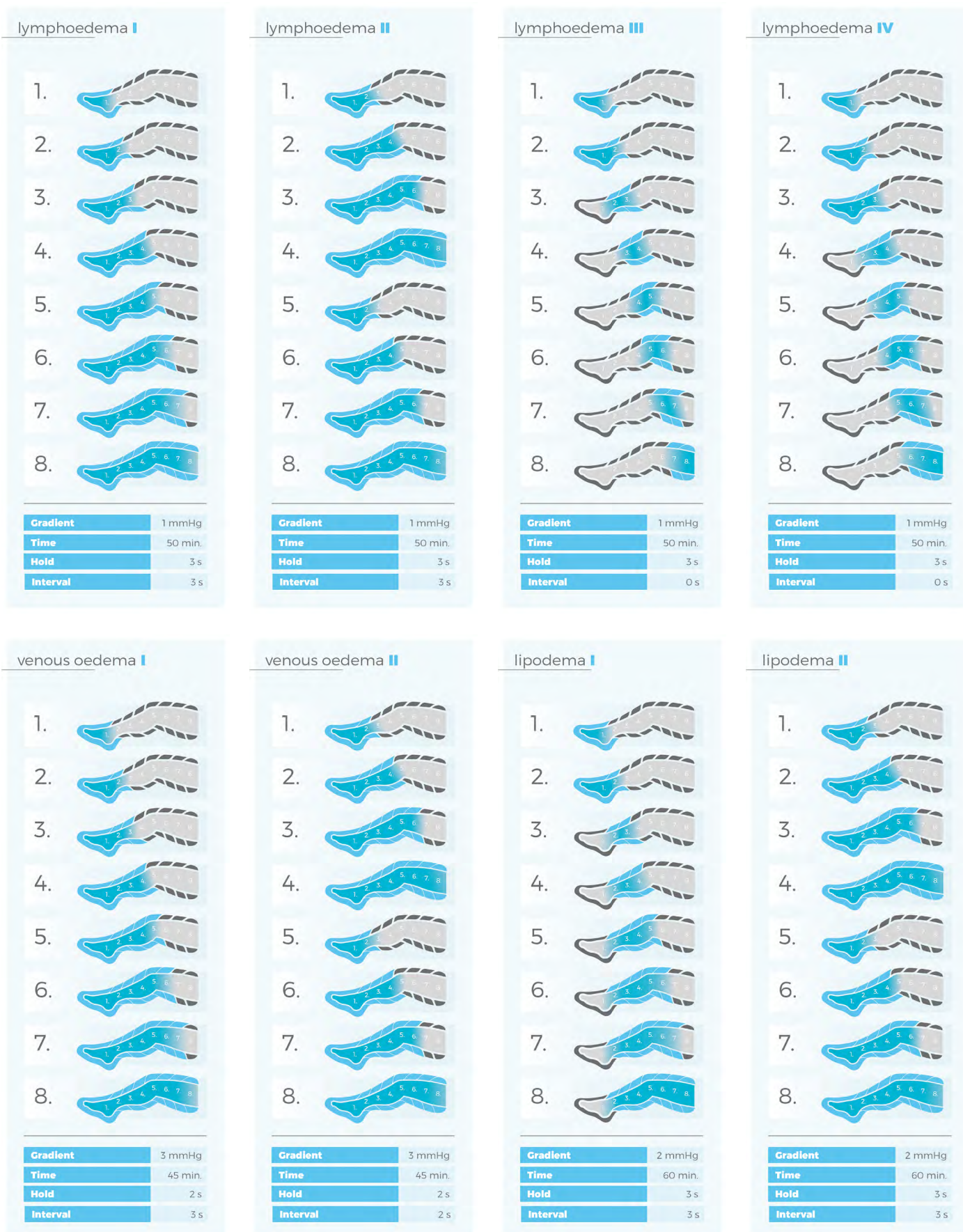


### post-injury therapy

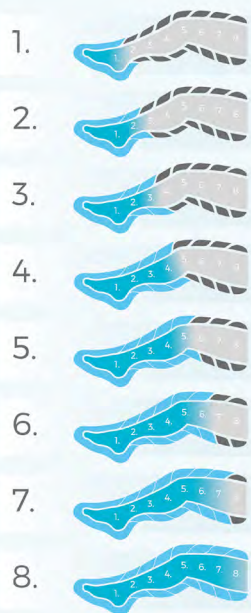


|          |         |
|----------|---------|
| Gradient | 4 mmHg  |
| Time     | 30 min. |
| Hold     | 2 s     |
| Interval | 2 s     |

### prevention, relaxation

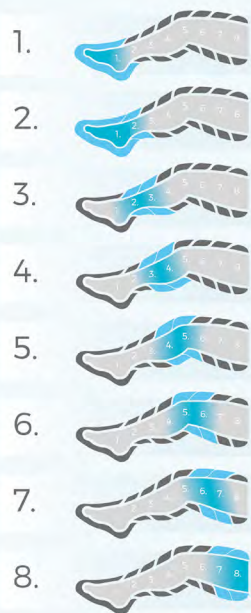


|          |         |
|----------|---------|
| Gradient | 0 mmHg  |
| Time     | 60 min. |
| Hold     | 2 s     |
| Interval | 2 s     |

### cellulite **advanced**

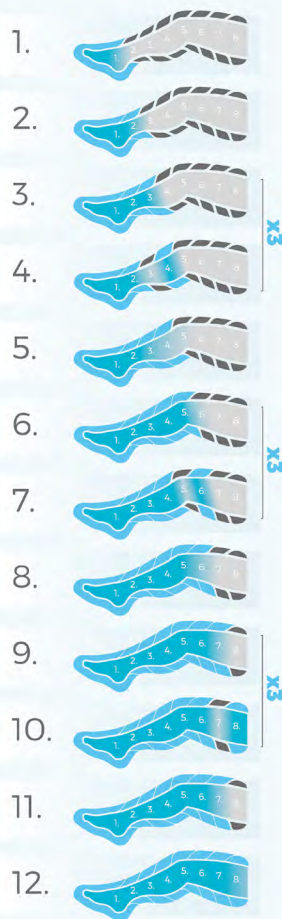


|          |         |
|----------|---------|
| Gradient | 1 mmHg  |
| Time     | 60 min. |
| Hold     | 1 s     |
| Interval | 2 s     |

### regeneration

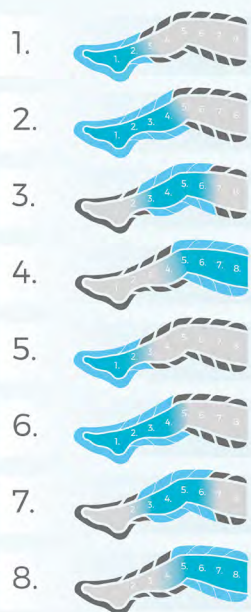

|          |         |
|----------|---------|
| Gradient | 4 mmHg  |
| Time     | 45 min. |
| Hold     | 2 s     |
| Interval | 2 s     |

### all-chamber massage

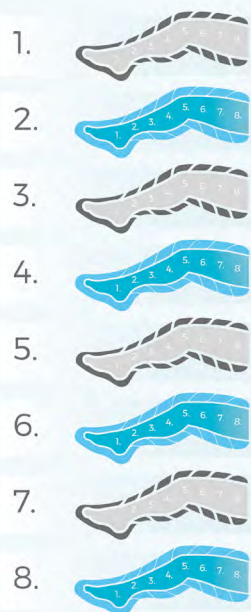

|          |         |
|----------|---------|
| Gradient | 0 mmHg  |
| Time     | 30 min. |
| Hold     | 2 s     |
| Interval | 3 s     |