



CARE PUMP

MOVE4
Sport line


Number of chambers	4
Max. pressure [mmHg]	220
Number of proprietary therapeutic pre-programs	6
Smart gradient' function	(mmHg, %, final pressure)
Precise regulation of pressure and time	in steps of 1, 5 or 10
Setting of individual pressure in each chamber	included
Number of independent channels	1
Adjustment of the speed of the treatment	1-5
Cuffs	overlapping system
Display	7" TN lcd color touch screen
Weight [kg]	2,5
Dimensions [mm]	290x130x130
Power supply	battery 21VDC/1A
Estimated battery life from full charge	4 h*
Warranty	1 year

*test carried out using the "Warm-up, prevention" program and the pressure of 100 mmHg

Accessories




Included:

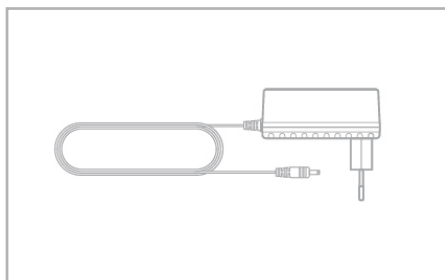


power supply

Optional:

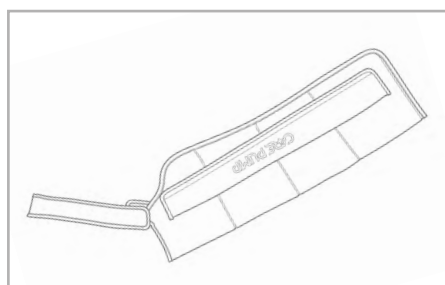


4-chamber arm cuff

CPARM4T

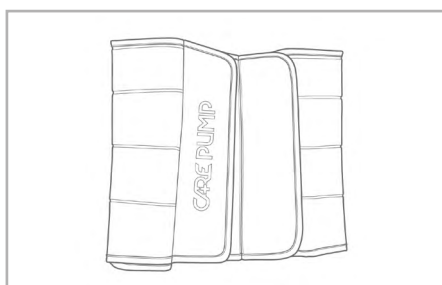


4-chamber waist cuff

CPWAIST4T

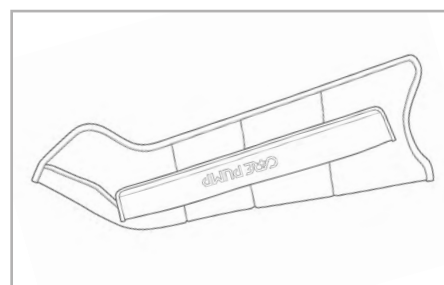

4-chamber left leg cuff

CPLEG4TL

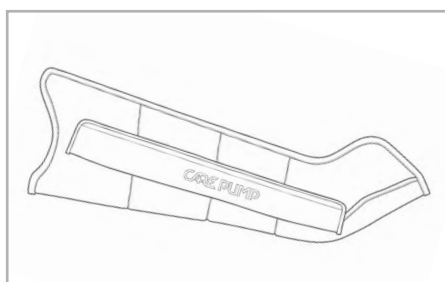


4-chamber right leg cuff

CPLEG4TR




4-chamber short pants

CPSHORTPANTS4T

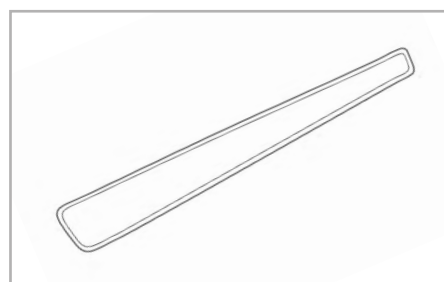


Extension zipper

CPLEGEXT4

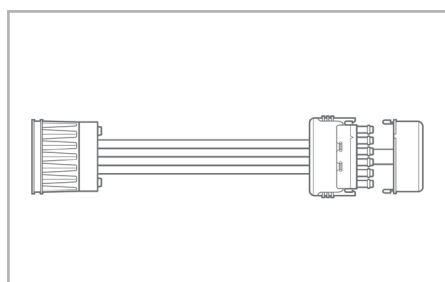


Double splitter

CPDOUBLECONNECTOR4

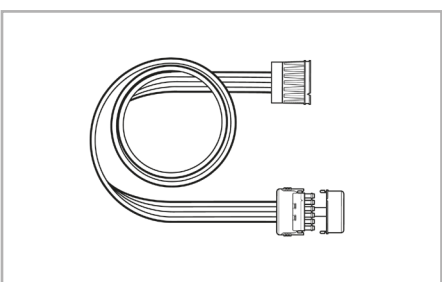

Double splitter (long - 150 cm)
for 4-chamber devices

CPDOUBLECONNECTOR4L

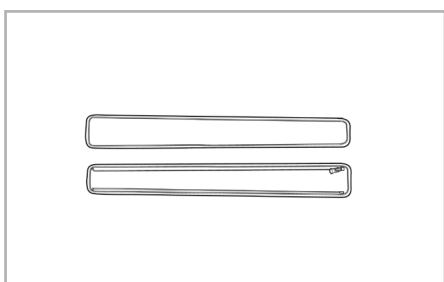


2 x Extension zippers
for 4-chamber short pants

CPSHORTPANTSEXT4

Therapeutic pre-programs




warm-up, prevention

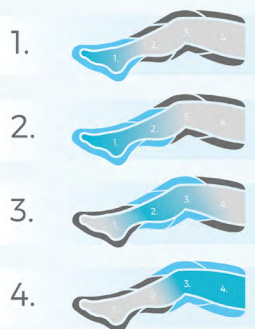


Gradient	0 mmHg
Time	60 min.
Hold	2 s
Interval	2 s

regeneration, wellness

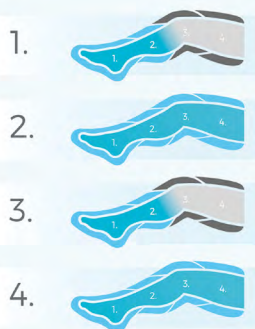


Gradient	4 mmHg
Time	45 min.
Hold	2 s
Interval	2 s

post-injury therapy

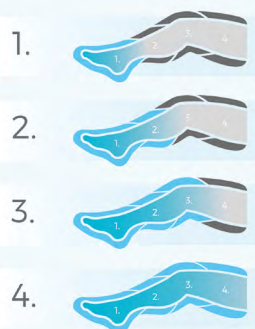


Gradient	4 mmHg
Time	30 min.
Hold	2 s
Interval	2 s

oedema therapy

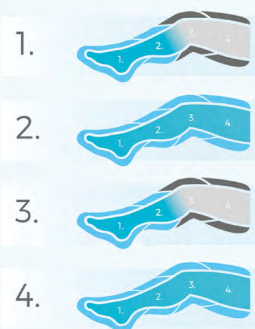

Gradient	1 mmHg
Time	50 min.
Hold	3 s
Interval	3 s

all-chamber massage

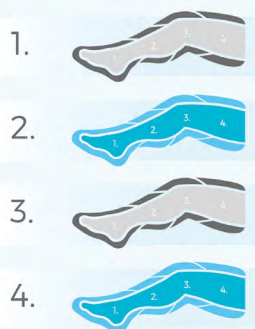

Gradient	0 mmHg
Time	30 min.
Hold	2 s
Interval	3 s

mixed mode

Possibility of combining any therapeutic pre-programs and selecting the number of repetitions for each of them.