



CARE PUMP

MOVE6
Sport line


Number of chambers	6
Max. pressure [mmHg]	220
Number of proprietary therapeutic pre-programs	6
Smart gradient' function	(mmHg, %, final pressure)
Precise regulation of pressure and time	in steps of 1, 5 or 10
Setting of individual pressure in each chamber	included
Number of independent channels	1
Adjustment of the speed of the treatment	1-5
Cuffs	overlapping system
Display	7" TN lcd color touch screen
Weight [kg]	2,8
Dimensions [mm]	290x130x130
Power supply	battery 21VDC/1A
Estimated battery life from full charge	4 h*
Warranty	1 year

*test carried out using the "Warm-up, prevention" program and the pressure of 100 mmHg

Accessories




Included:

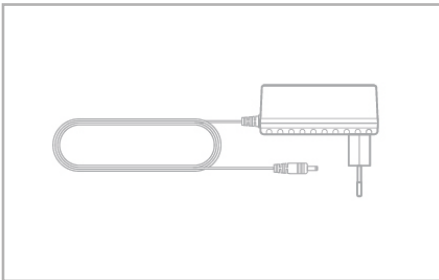


power supply

Optional:

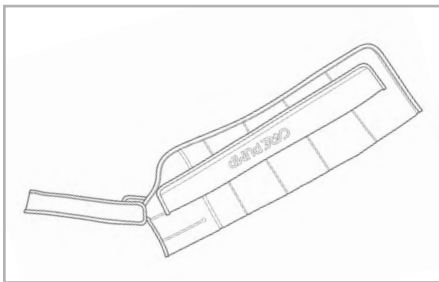


6-chamber arm cuff

CPARM6T

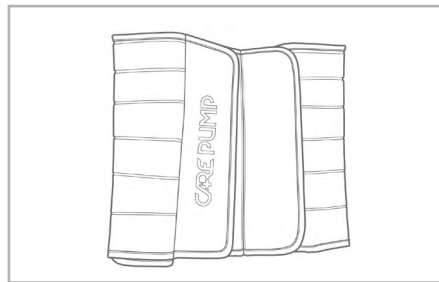


6-chamber waist cuff

CPWAIST6T

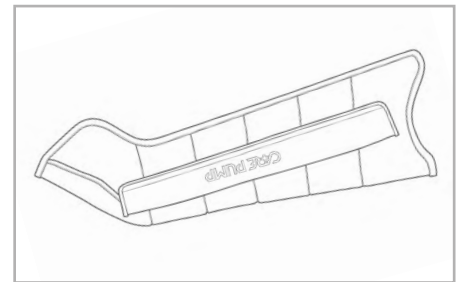

6-chamber left leg cuff

CPLEG6TL

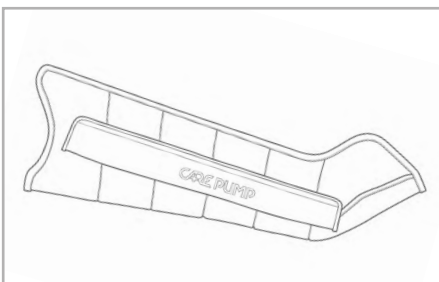


6-chamber right leg cuff

CPLEG6TR




6-chamber short pants

CPSHORTPANTS6T

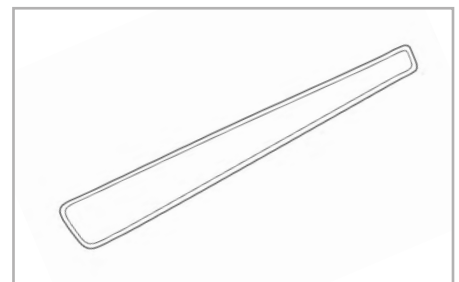


Extension zipper

CPLEGEXT6

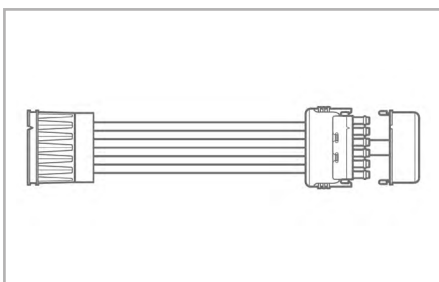


Double splitter

CPDOUBLECONNECTOR6

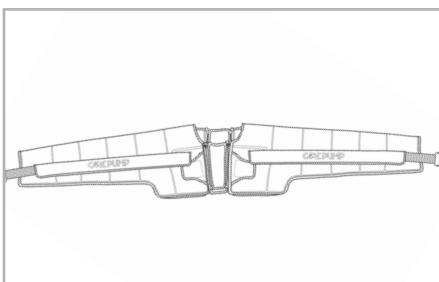


6-chamber double cuff for arms, shoulders and chest

CPDOUBLEARM6T

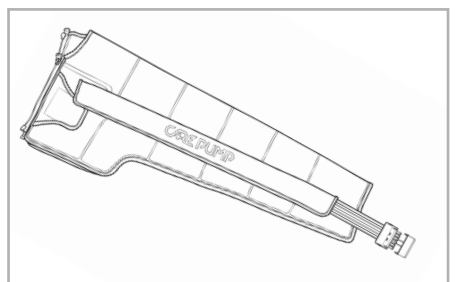


6-chamber left arm, shoulder and chest cuff

CPARMSB6TL

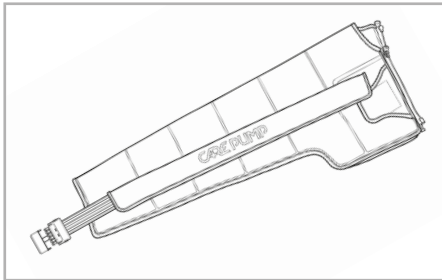


6-chamber right arm, shoulder and chest cuff

CPARMSB6TR

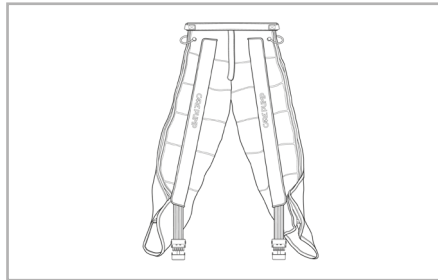


6-chamber full pants

CPLEG6TL

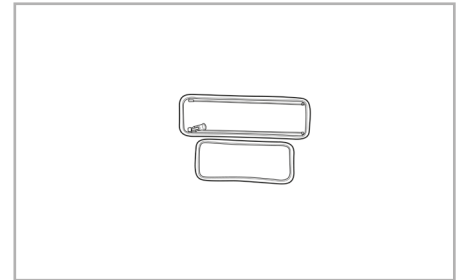


2 x Extension zippers for 6-chamber double cuff for arms, shoulders and chest

CPDOUBLEARMEXT6

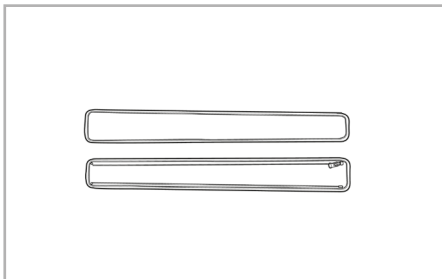


2 x Extension zippers for 6-chamber short pants

CPSHORTPANTSEXT6

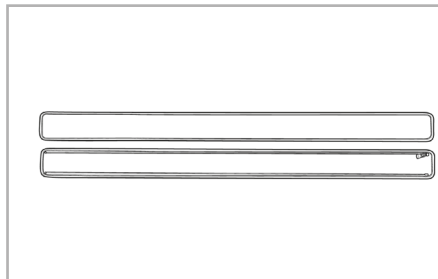

2 x Extension zippers for 6-chamber full pants

CPLONGPANTSEXT6

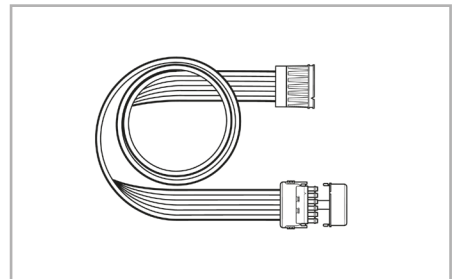


Double splitter (long - 150 cm) for 6-chamber devices

CPDOUBLECONNECTOR6L

Therapeutic pre-programs




warm-up, prevention

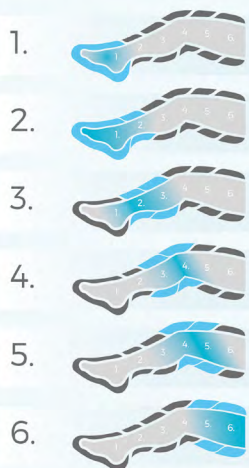


Gradient	0 mmHg
Time	60 min.
Hold	2 s
Interval	2 s

regeneration, wellness

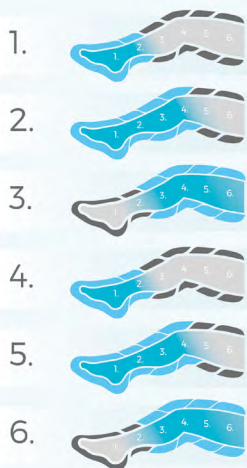


Gradient	4 mmHg
Time	45 min.
Hold	2 s
Interval	2 s

post-injury therapy

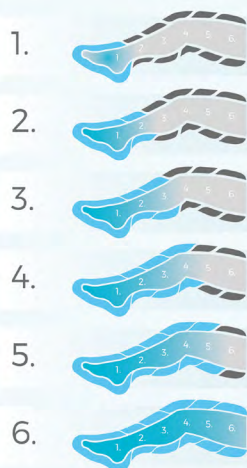


Gradient	4 mmHg
Time	30 min.
Hold	2 s
Interval	2 s

oedema therapy

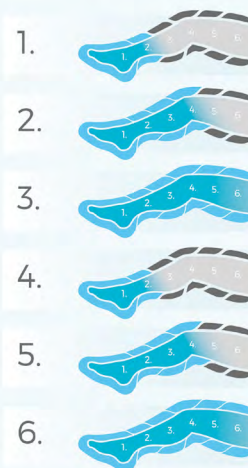

Gradient	1 mmHg
Time	50 min.
Hold	3 s
Interval	3 s

all-chamber massage

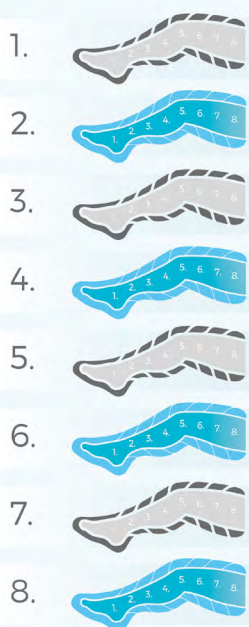

mixed mode

Possibility of combining any therapeutic pre-programs and selecting the number of repetitions for each of them.